## WIG-CAP

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### INSTRUCTION MANUAL

#### Brief description:

A Wig-Cap fixes the natural hair and makes it fit tightly to the head. It also forms a grippy surface for the wig or a turban and thus provides a better hold. Due to the close-fitting hair, the usual head shape is maintained even with a wig and it creates a natural appearance. A Wig-Cap can also be worn without hair. It is often used when trying on a wig to ensure a hygienic fitting.

The Wig-Cap is available in several designs. Due to the very fine material, a Wig-Cap is intended for single use. With careful handling, the Wig-Cap can also be used several times.

#### Wig-Cap coarse open:

The Wig-Cap coarse open is especially suitable for use with natural hair. The coarse mesh fabric is extremely stretchy and adapts to the hairstyle under the wig. Thus, the hair is fixed and kept in shape.

Through the coarse meshes, the hair can be attached to or woven into the Wig-Cap using hairpins. Through the opening at the back of the head, the own hair can be led out, alternatively, the Wig-Cap can be closed again by twisting the opening. The net can be cut in without tearing it open further.

The Wig-Cap is available individually or in packs of 30 pieces each.

#### Wig-Cap coarse closed:

The Wig-Cap is extremely stretchy and adapts to the hairstyle or head shape under the wig. Thus, the hair is fixed and kept in shape.

Through the coarse meshes, the hair can be attached to or woven into the Wig-Cap using hairpins. The mesh can be cut in without tearing further.

#### Wig-Cap fine mesh closed:

The Wig-Cap fixes the natural hair or protects the sensitive scalp under the wig. The material is stretchy and adapts to any head size.

The Wig-Cap is available individually or in packs of 30 pieces each.

#### Application instructions:

Learn more on [Aderans.eu](https://www.aderans.eu).

Put on the Wig-Cap like a beanie and pull it back behind the hairline. When putting on the Wig-Cap, it is helpful to gather your own hair in a braid beforehand or to lead it forward overhead so that all the hair is bundled under the Wig-Cap.

The stretchable material prevents hair breakage under the Wig-Cap.

The Wig-Cap can be trimmed to the desired shape if needed.

**Further notes:**

Additional clips and attachments may cause pressure points.

The Wig-Cap can be washed by hand or on the gentle cycle in the washing machine.

After washing, gently pull into shape and allow to dry.

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