SENTOO

COLLECTION

INSTRUCTION FOR USE

SENTOO
COLLECTION

БЪЛГАРСКИ	ИНСТРУКЦИИ ЗА УПОТРЕБА	4
ČEŠTINA	NÁVOD K POUŽITÍ	8
DANSK	INSTRUKTIONER FOR BRUG	12
DEUTSCH	GEBRAUCHSANWEISUNG	16
ENGLISH	INSTRUCTION FOR USE	20
ESPAÑOL	INSTRUCCIONES DE USO	24
FRANÇAIS	INSTRUCTIONS D'EMPLOI	28
HRVATSKI	UPUTE ZA UPORABU	32
NEDERLANDS	GEbruikSAANWIJZING	36
NORSK	INSTRUKSJONER FOR BRUK	40
PORTUGUÊS	INSTRUÇÕES DE USO	44
SUOMI	HOITO-OHJEET	48
SVENSKA	BRUKSANVISNING	52

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INSTRUCTION FOR USE

PROVIDING THE BEST CARE FOR OPTIMUM AND LONG-LASTING RESULTS.

Please read these care instructions carefully and keep them in a safe place for future reference.

SCOPE OF DELIVERY

Upon receiving your product, please check that the contents are correct and contain the following:

• Wig • Instructions for use • Hair net • Storage box.

INTENDED USE

The Medical Wigs are intended to be used to improve the Quality of Life (QOL) for patients suffering from partial or total hair loss.

INDICATIONS

People suffering from reduced Quality of Life (QOL) resulting from partial or total hair loss and older than 3 years of age.

CONTRAINDICATIONS

You should avoid wearing Medical Wigs or hairpieces if you have known intolerances or allergies to any of their components, particularly polyurethane or nylon.

You should also avoid wearing a wig or hairpiece if you have skin irritations or open wounds in the area where it will be worn.

USE ENVIRONMENT

Home environment, outdoor or indoor.

SAFETY INSTRUCTIONS

Always keep the wig away from heat, hot water, hair dryers, curling irons, open ovens and other sources of heat or of extreme light. Human hair wigs and heat-resistant fibres can be blow-dried and styled, but only at a low temperature. Do not expose the wig to excessive heat. Do not use highly odorous or corrosive cleaning fluids. Always follow the recommended care instructions for synthetic or human hair carefully. The type of hair used in the wig is labelled on the hangtag.

In rare cases, material sensitivities may occur. If you experience any sensitivity or adverse reaction, please contact your local wig specialist. Any serious incident that occurs in relation to the wig should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

INSTRUCTIONS

If required, please remove the hair net before wearing the wig. When putting on the wig, please be extremely careful and ensure it fits properly. The label is positioned at the back of the head.

CARE INSTRUCTIONS | SYNTHETIC HAIR

Contact with hot water, open ovens, radiant heaters and external light sources must be avoided.

1 | SHAMPOO Remove adhesive residue, if present. Gently brush through the wig. For long hair, start at the ends and gradually work your way up. Place a small amount of some Aderans Shampoo in a bowl of lukewarm water and immerse the wig. Do not rub or wring. The shampoo is self-cleaning.

2 | RINSE Rinse the hair fibres thoroughly by running plenty of lukewarm water, working from the roots to the ends, until all the shampoo residue has been removed. Do not use circular movements. Gently pat the hair fibres dry with a soft towel. Avoid unnecessary rubbing or friction.

3 | CONDITION Place a small amount of Aderans synthetic hair Conditioner in a bowl of lukewarm water and immerse the wig. Do not rub or wring the wig. The conditioner is self-acting. Leave the wig in the solution for approximately 3-5 minutes. Rinse thoroughly as described in step 2.

4 | DRY Gently wrap the wig in a towel and carefully squeeze out the excess water from the roots to the tips. Do not rub. Place the wig on a wig stand. Do not twist or stretch the wig or its base. Do not comb or brush the wig while it is wet. Allow the wig to air dry naturally.

5 | STYLING We recommend having the wig styled by a professional if you have any specific requirements. The wig can be damaged by the use of heated styling devices and excessive styling.

6 | WARNINGS AND PRECAUTIONS

- Never brush or comb the wig while it is wet.
- Never expose the wig to direct heat.
- Do not chemically treat the wig.
- Do not style the wig using hair dryers, heated styling tools, curling irons, or flat irons.
- Do not stretch or twist the wig.
- Don't stretch the wig excessively when putting it onto the head.

CARE INSTRUCTIONS | HIGH HEAT FIBRE

To ensure you enjoy your Sentoo high heat fibre product for as long as possible, it is important to adhere to the manufacturer's care instructions. Following the manufacturer's care instructions carefully is essential to ensure your Sentoo high heat fibre product receives the care it needs to maintain its quality, durability, appearance, and feel of your high-heat fibre product.

Use of Aderans products: It is important that you only use the cleaning and care products from the Aderans range as these are specially formulated for use on high heat fibres. These products contain specially formulated ingredients that balance the moisture content of the fibre, helping to maintain its healthy appearance and extend the lifespan of your Sentoo high heat product.

Regular and proper care: It is important to provide regular care to maintain your high heat fibre product. When caring for a high heat fibre product at home, always follow the manufacturer's instructions. Professional care by a qualified hair specialist will ensure that the hair fibre maintains its natural appearance and silky soft feel month after month.

1 | CLEANSE WITH REVITALISING SHAMPOO Gently comb through your high heat fibre product, carefully removing any knots or light tangles from the ends before gradually working your way up towards the root.

Mix a small amount of *Revitalising Wash Shampoo* with lukewarm water in a sink or bowl.

Immerse the high heat fibre product into the shampoo and water solution, leave it for 3 to 5 minutes whilst the shampoo begins to cleanse the product.

Gently swirl the high heat fibre product in circular motions within the water bath for a few seconds. Remove the product from the bowl and rinse gently with clean, lukewarm water.

2 | CONDITION WITH DETANGLING CREAM Mix a small amount of *Detangling Cream* with lukewarm water in a bowl.

Immerse the high heat fibre product into the solution and leave it on for 3 to 5 minutes to condition the fibres.

Gently swirl the high-heat fibre product in the conditioner solution for a few seconds.

Remove the product from the bowl and rinse gently with clean, lukewarm water.

3 | DRYING Wrap the high-heat fibre product in a clean, dry towel and gently pat dry. Do not massage, rub, or apply unnecessary friction.

4 | DAILY PROTECTION WITH MOISTURE PROTECTION Apply *Moisture Protection* spray to damp hair and allow the high-heat fibre product to air dry naturally.

5 | BRUSHING Spray the wig with *Moisture Protection* spray before and after brushing to make brushing and styling easier. Only brush the hair when it is dry to avoid loosening the knots.

Brush the hair in sections, starting at the ends and carefully working your way up to the roots. Avoid applying excessive force or pulling, as this may cause the knots to loosen.

6 | (HEAT) STYLING Use a hairdryer on a low heat setting to dry the high-heat fibre product. Be careful not to dry it completely, to avoid removing too much moisture from the fibres.

When heat styling with straighteners, curling irons or curling tongs, make sure that the temperature setting of the styling appliances does **not exceed 140°C**.

Always ensure the fibre is wet before applying heat. After brushing and styling, it is advisable to spray on the *Moisture Protection* spray again. For additional hair texture and flexible hold, use *Aderans Styling Gel* or *Styling Cream*. Use the *Aderans Hairspray* to set and hold the style.

7 | IMPORTANT NOTES

- It is advisable to repeat the cleaning and care routine described above every 7 to 10 days to help maintain the moisture content of the high-heat fibre product.
- Use only the recommended cleaning and care products to maintain the validity of your high heat fibre product guarantee.
- Only use temperature-regulated heat styling appliances.

CARE INSTRUCTIONS | HUMAN HAIR

Where possible, avoid exposing the wig to direct sunlight, chlorine, and salt water.

1 | SHAMPOO Gently comb through the wig using a wide-toothed comb, to remove any knots. Start at the ends of the hair and gradually work your way up towards the roots to avoid pulling the hair.

Place the wig on a wig stand, making sure the wig is taut and wrinkle-free, without stretching or twisting the wig. Securing the wig properly to the wig stand before washing is very important. This helps prevent the hair from slipping through the base and loosening the knots.

Dissolve a hazelnut to walnut-sized amount (depending on hair length) of *Aderans Hydrating Shampoo* in a tub or sink with lukewarm water. Immerse the wig head in the water several times, making sure the hair is thoroughly soaked. Gently move the wig back and forth in the water for about 30 seconds. Alternatively, apply the Hydrating Shampoo directly on the hair, working from the top of the head towards the end. If there is heavier residues, gently massage in shampoo. Leave on for about 10 minutes. Do not rub or wring. The *Aderans Hydrating Shampoo* is self-cleaning. Prolonged exposure can cause the knots to swell and thus lead to hair loss.

2 | RINSE Rinse the wig thoroughly by running plenty of lukewarm water from the roots to the tips until all shampoo residue has been removed. Do not use circular movements. Gently pat the hair fibers dry with a soft towel – avoid unnecessary friction.

3 | CONDITION Apply a hazelnut-sized amount of *Aderans Moisturising Conditioner* to the lengths and ends of the hair. Beginning at the nape area and working towards the crown area. Gently run your hands through the wig to distribute the wig conditioner evenly. Do not apply the conditioner too close to the knots, as this may cause them to loosen (keep about 2.5 cm away). Leave on for about 10 minutes.

4 | RINSE Rinse the wig thoroughly with plenty of lukewarm water, working from the roots to the ends, until all conditioner residue has been removed. Do not use circular movements. Gently pat the hair dry with a soft towel. Avoid unnecessary friction.

5 | DRY Remove the wig from the wig stand. Gently wrap the hair in a towel and carefully squeeze out excess water from the roots to the tips. Do not rub. Briefly blow-dry the wig from the inside (using low heat setting), then place the wig on a wig stand. Do not twist or stretch the wig. Gently comb through the hair using a wide-toothed comb.

6 | STYLING We recommend having the wig styled by a professional if you have specific requirements. If desired, blow-dry the wig gently at a low heat. The wig can be damaged by the excessive use of styling devices and excessive styling.

7 | WARNINGS AND PRECAUTIONS

- Do not stretch or twist the wig.
- Avoid excessive use of hair cleaning products.
- Always comb the wig starting at the ends and gradually work your way up towards the roots.
- Only blow-dry or style the wig when it is almost dry.
- Do not treat the wig chemically.

FURTHER USE OF WIG

Due to hygiene reasons, the wig is intended for personal use only. Once you no longer require the wig, dispose of it in accordance with the disposal instructions below.

DISPOSAL INSTRUCTIONS

The wig can be disposed of with the household waste, as long as there is no risk that it has been exposed to any infectious disease. This product must not be disposed of in the organic waste or recycling.

TERMS OF DELIVERY AND GUARANTEE

Upon receipt, the customer must inspect the goods immediately for defects or damage. In the event of a complaint, proof of purchase must be provided by means of a valid receipt. The warranty does not cover natural wear and tear and will be voided due to negligence or incorrect care. The warranty will be void if the customer modifies the product or assigns a third party to modify the product.

	LEGAL MANUFACTURER Aderans Europe B.V. Eglantierbaan 16, 2908 LV Capelle aan den IJssel, Netherlands info@aderanseurope.com www.aderans.eu		CE Mark Medical device according to EU MDR 2017/745
	Model number		Consult instructions for use
	Batch number including manufacturing date YYYY-MM		For storage: Keep away from sunlight
	Medical Device		Contains biological material of human origin

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